



## AUDIOLOGIST IMPRESSION GUIDE

**When meeting the Audiologist for the ear impressions make sure you have this guide with you.**

**It will help him have a better understanding of what we need in order to create the best custom IEMs for you.**

We require a **full impression of each ear**: include the full helix, crus of the helix, tragus and antitragus. The impression must be taken completely past the second bend of the ear canal.

The **ear canal** must be examined for any obtrusions or inflammations (if any exist we recommend not to continue with the procedure).

Insert an **oto-block slightly beyond the second directional bend**.

A **Byte Block** is recommended to have the patient's jaw open about 2,5 cm while the impression cures. With this method will have a more secure fitment and seal.

To have just the right thing, the impression should be **made of silicone material** and we recommend one with high viscosity.

Let us know when your impressions are ready!

[iem@lx-ear.ro](mailto:iem@lx-ear.ro) | +40 765 452 556